MEDISANA®

GB Blood Pressure Monitor BW 310

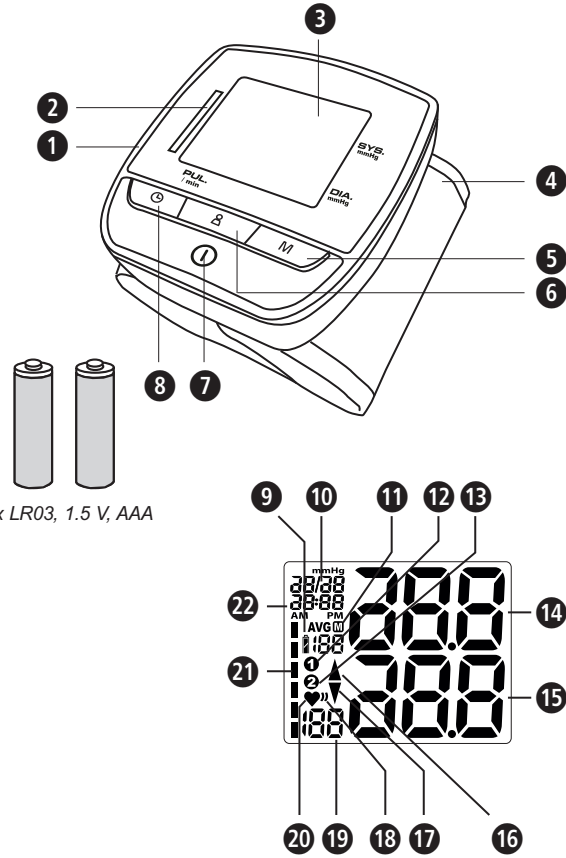
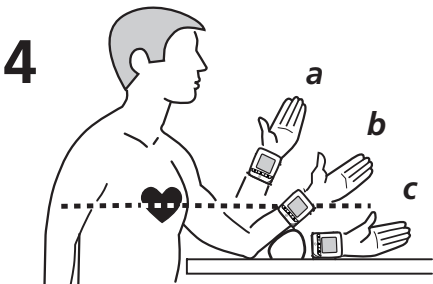
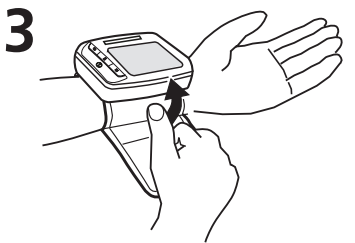
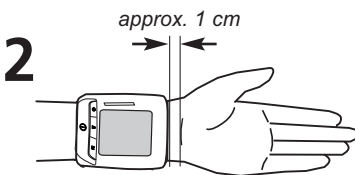
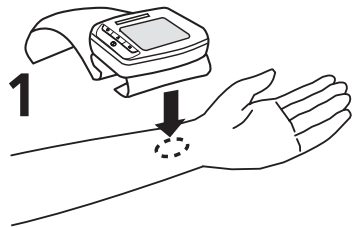
Instruction Manual

Please read carefully!

GB Unit and LC-Display

Proper use

- This fully automatic electronic blood pressure monitor is intended for measuring the blood pressure at home. It is a non-invasive blood pressure measurement system to measure the diastolic and systolic blood pressure and pulse of an adult using a non-invasive technique by means of a cuff, which needs to be fitted on the wrist.

Contraindications

- The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- Persons who suffer from arrhythmia, diabetes, cardiovascular problems or who have had a stroke should consult their doctor before using the device.

Common factors of wrong measurements

- Please keep quiet for 5-10 minutes and avoid eating, drinking alcohol, smoking, exercising and bathing before taking a measurement. All these factors will influence the measurement result.
- Remove watches or other spruce from your wrist before measurement.
- Always measure on the same wrist (normally left).
- Measurements should be taken regularly at the same time of each day, as the blood pressure varies even during the day.
- Any effort to support the arm during measurement may increase the measured blood pressure.
- Make sure, you are in a comfortable, relaxed position and do not move or constrict your muscles during measurement. Use a cushion to support your arm if necessary.
- If the wrist is not at heart level, a false reading may be obtained.
- A loose or open cuff causes false readings.
- With repeated measurements, blood accumulates in the wrist, what may lead to false readings. Consecutive blood pressure measurements should be carried out after a 5-minute pause or after the arm has been held up in order to allow the accumulated blood to flow normally.

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
 - Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
 - Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
 - The unit may not be used to check the heart rate of a pacemaker.
 - Like on any other oscillometric blood pressure measurement devices, certain medical conditions can affect the measurement accuracy, among others: cardiac arrhythmias, low blood pressure, circulatory disorders, states of shock, diabetes, pregnancy, pre-eclampsy etc. Always consult a physician before you use the device.
 - This device is not suitable to provide a medical diagnosis. The measurement results are for reference only. Contact your physician to get information about treatment and medicine, which are necessary for you.
 - The arrhythmia function does not replace a professional cardiac examination. Check with your doctor if any treatment or medical action is necessary for you.
 - A heartbeat that varies by 25% or more from the average of all heartbeats during a measurement may cause difficulties to determine the correct blood pressure.
 - The "Blood pressure classification WHO" is not suitable to replace a medical diagnosis. Ask your physician to individually evaluate your blood pressure.
 - If you experience discomfort during a measurement, such as pain in the wrist or other complaints or in case the device does not stop to inflate the cuff, press the **○** - button **7** to release the air immediately from the cuff. Loosen the cuff and remove it from your wrist.
 - Blood pressure measurements in high repetition rates may lead to unwanted side effects, e.g. nerval compression or blood clots.
 - Blood pressure measurements - especially if executed frequently - may lead to temporary marks on the skin. These marks may be visible for several days in individual cases. If necessary, contact your physician to get more information.
 - This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
 - This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons.
 - Do not allow children to use the unit. Medical products are not toys!
 - The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.
 - The user needs to check that the equipment functions properly and that it is in proper working condition before being used.
 - The device needs to be used in accordance to the specified ambient conditions (see "Technical specifications") only. Otherwise the accuracy of readings might be affected.
 - The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves, as this could cause functional defects or incorrect measurements.
 - Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.
 - Do not take measurements, if any other equipment is applied to the same limb for monitoring. This could lead to disturbances or failures.
 - After a lymph node removal operation a physician should be consulted, before a blood pressure measurement is executed with this device.
 - Never apply the cuff over hurt skin or in case of an inserted catheter.
 - Do not apply the cuff on any other part of the body than the left wrist.
 - Before applying the cuff, make sure there are no wrinkles in the cuff, as these could cause bruises.
 - The cuff size should be chosen according to the wrist circumference. Non fitting cuffs may lead to inaccurate measurements.
 - The cuff must not be applied over clothes as this may lead to injuries or inaccurate readings.
 - Do not kink or fold the cuff and do not pull it over sharp edges.
 - No modification of this unit/device is allowed.
 - Please do not attempt to repair the unit yourself in the event of malfunctions, since this will void all warranty claims. Only have repairs carried out by authorised service centres.
 - Protect the unit against moisture. If liquid does manage to penetrate into the unit, remove the batteries immediately and do not continue to use the unit. In this case contact your specialist dealer or notify us directly.
 - Extreme temperatures, humidity or altitudes may impair the performance of the device.
 - Please do not use a dilution agent, alcohol or petrol to clean the unit.
 - Do not hit the device heavily or let it fall down from a high place.
 - Do not insert any objects into the device openings and never push the buttons with heavy force or with the help of peak objects. This may lead to damages of the device.
 - If the unit is not going to be used for a long period, please remove the batteries.
- #### SAFETY NOTES FOR BATTERIES
- Do not disassemble batteries!
 - Replace all batteries if the unit display shows the low battery symbol.
 - Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
 - Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
 - If a battery has been swallowed, seek medical attention immediately!
 - Replace all of the batteries simultaneously!
 - Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
 - Insert the batteries correctly, observing the polarity!
 - Keep batteries out of children's reach!
 - Do not attempt to recharge batteries! **There is a danger of explosion!**
 - Do not short circuit! **There is a danger of explosion!**
 - Do not throw into a fire! **There is a danger of explosion!**
 - Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Unit and LCD display

- 1** Lid of the Battery Compartment
- 2** Blood Pressure Indicator (green - yellow - orange - red)
- 3** LC-display
- 4** Wrist cuff
- 5** **M** - button (Memory Recall)
- 6** **Δ** - button (User Setting)
- 7** **○** - button (START/STOP)
- 8** **○** - button (Clock/Date)
- 9** Change Battery symbol
- 10** Average value
- 11** Memory Location number
- 12** User memory 1
- 13** User memory 2
- 14** Display of Systolic Pressure
- 15** Display of Diastolic Pressure
- 16** Inflation symbol
- 17** Deflation symbol
- 18** Arrhythmia display (arrhythmia cordis)
- 19** Display of Pulse Rate
- 20** Pulse symbol
- 21** Blood Pressure Indicator
- 22** Display of Time / Date

What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

How is blood pressure measured?

The **MEDISANA BW 310** is a blood pressure unit which is used to measure blood pressure from the wrist. The measurement is carried out by a micro-processor, which, via a pressure sensor, measures the vibrations resulting from the inflation and deflation of the cuff over the artery.

Blood pressure classification WHO

These values have been specified by the World Health Organisation (WHO) without making any allowance for age.

Low blood pressure	systolic <100	diastolic <60
Normal blood pressure	(green display area 2 / 4)	systolic 100 - 139 diastolic 60 - 89

Forms of high blood pressure / hypertension

Mild hypertension	(yellow display area 2 / 4)	systolic 140 – 159 diastolic 90 – 99
Moderate hypertension	(orange display area 2 / 4)	systolic 160 – 179 diastolic 100 – 109
Severe hypertension	(red display area 2 / 4)	systolic ≥ 180 diastolic ≥ 110

WARNING
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Starting up

Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment **1** is located on the left hand side of the unit. Open it by pushing the cover outward and remove it. Insert the 2 AAA LR03 type 1.5 V batteries supplied. Ensure correct polarity when inserting the batteries (as marked on the battery compartment cover). Close the battery compartment. Replace the batteries immediately, if the partial change battery symbol **9** flashes on the display **3** and "E6" is appearing at the same time or if nothing appears on the display after the unit has been switched on. If the partial change battery symbol **9** appears on the display constantly together with "E6", the batteries are already in poor condition. If the full change battery symbol **9** appears, the batteries are in good condition.

Settings

After inserting the batteries, time and date need to be adjusted correctly. The device automatically switches to settings mode after insertion of the batteries. If you want to get to settings mode manually, press and hold the **○** - button **8** for approx. 3 seconds after switching on the unit.

1. Adjustment of the year:

The year figure starts to flash for setting. Keep pressing the **M** - button **5** until the required year appears. Press **○** - button **8** to confirm the year setting. Afterwards, you will enter the month/date setting mode.

2. Month / date setting:

The month figure starts to flash for setting. Keep pressing the **M** - button **5** until the required month appears. Press **○** - button **8** to confirm the month setting. Afterwards, the day needs to be set. As with the month setting, keep pressing the **M** - button **5** until the required day appears. Press **○** - button **8** to confirm the day setting. Afterwards, you will enter the time setting mode.

3. Time setting:

The hour figure starts to flash for setting. Keep pressing the **M** - button **5** until the required hour appears. Press **○** - button **8** to confirm the hour setting and proceed in the same way for minute setting. After this, all settings are completed. The display shows the actual time and date. Press **○** - button **7** to switch off the device. If the batteries are changed, the set information is lost and needs to be entered again.

Setting up the user memory

The **MEDISANA** wrist blood pressure monitor **BW 310** gives you the opportunity to assign readings to one of two user memories, each of which provides 60 memory slots. After switching on the device (press **○** - button **8** to switch the device on), press **Δ** - button **6** to select between user 1 **1** and user 2 **2**. Your selection will be saved by the device and will be considered in next blood pressure measurement respectively when the measurement result is saved. Your selection is valid until you make another selection. After selecting the user, press **○** - button **7** to switch off the device.

Fitting the cuff

- Attach the clean cuff to the left wrist which is free from clothing with the palm of the hand facing upwards (fig.1).
- There should be a gap of approx. 1 cm between the cuff and the palm of the hand (fig.2).
- Press the Velcro tape firmly around the wrist ensuring that it is not so tight that it could falsify the measured result (fig.3).

Correct position for taking readings

- Carry out the measurement whilst seated.
- Relax your arm and place it comfortably, on a table for instance.
- Raise the wrist so that the cuff of the monitor is at heart level (fig. 4, position **a** = too high, **b** = correct, **c** = too low).
- Keep still during the measurement: Do not move and do not talk as this may alter the measurements.

Taking a blood pressure measurement

- After the cuff has been appropriately positioned, the measurement can begin:
- Switch the unit on by pressing the **○** - button **7**. The measurement will also start immediately.
 - The unit is ready for measurement and the average value of the last 3 measurements and the last memory slot number shortly appear. The device automatically inflates the cuff slowly in order to measure your blood pressure.
 - The rising pressure in the cuff is shown on the display. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol **♥** **20** starts to flash on the display.
 - When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display **3**. The blood pressure indicator **2** is shown next to the relevant coloured bar depending on the WHO blood pressure classification. If the unit has detected an irregular heartbeat, the arrhythmia display **♥** **18** will also be shown.

WARNING
Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

- The readings are automatically saved in the selected memory (**1** or **2**). Up to 60 measured values with date and time can be stored in each memory.
- The measurement readings remain on the display. The unit switches off automatically after approx. 2 minutes if no buttons are pressed. The unit may also be switched off using the **○** - button **7**.

Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (eg. the patient feels unwell) the **○** - button **7** can be pressed at any time. The device immediately decrease the cuff pressure automatically.

Display stored values

This unit features 2 separate memories, each with a capacity of 60 memory slots. Results are automatically stored in the memory selected by the user. Press the **○** - button **8** when power off to power on the device. To call up the measured values stored, press the **Δ** - button **6** to select the user (**1** or **2**) and then the **M** - button **5**. The average values of the last 3 measurements appear **10** on the display, together with "AVG". Pressing the **M** - button **5** again, the last measurement value will be shown. Repeatedly pressing the **M** - button **5** displays the respective values measured previously. If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 2 min. in memory recall mode. Memory recall mode can be exited at any time by pressing the **○** button **7** which will also switch the unit off. The oldest value is deleted if 60 measured values have been stored in the memory and a new value is saved.

Delete saved values

If you want to delete the record of a saved measurement, first follow the instructions under "Display stored values" to display the respective record. Now press and hold the **M** - button **5**, until **CL** and "--" appears. Press the **Δ** - button **6** to confirm the deletion. If you are sure that you want to permanently remove all stored memories, follow the instructions under "Display stored values" to display the average values of last 3 measurements **10** ("AVG" lights up). Press and hold the **M** - button **5**, until **CL** and "--" appears. Press the **Δ** - button **6** to confirm the deletion. After each successful deletion **CL** and "00" appears on the display.

Error messages and error remedying

Error indicators

The following symbol will appear on the display when measuring abnormal:

Symbol	Cause	Correction
E-1	Cuff not fitted properly	Put on the cuff correctly. Repeat the measurement in the correct way.
E-2	Movement or speaking during measurement	Repeat the measurement in the correct way after relaxing 30 minutes. Do not speak or move during the measurement.
E-3	Error during inflation process	Put on the cuff correctly. Repeat the measurement.
E-4	Measurement can not be proceeded or measurement failed	A weak pulse signal may be the cause for measurement problems. Repeat the measurement after relaxing 30 minutes. If you get unusual readings for 3 times, please contact your doctor.
E-5	Too high cuff pressure	Pressure above 300 mmHg. Contact your doctor.
E-6	Weak batteries	See chapter "Insert / replace batteries"
-	Device constantly re-pumps the cuff	Remove the batteries and insert new ones. Repeat the measurement.

Please contact the manufacturer resp. the service centre if you can't solve the problem. Do not disassemble the unit by yourself.

Cleaning and care

Remove the batteries before cleaning the unit. Clean the unit and the cuff using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents, alcohol, naphtha, thinner or gasoline etc.. Never immerse the unit or any component in water. Be cautious not to get any moisture in the main unit. Do not use the unit again until it is completely dry. Only inflate the cuff when it is in position around the wrist. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not subject the unit to extreme hot or cold temperature. Keep the unit in the original packing when not in use. Store the unit in a clean and dry location.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard for non-invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE 0297". The blood pressure monitor corresponds to European standards EN 1060-1 and EN 1060-3. The specifications of EU Guideline "93/42/EEC of the Council Directive dated 14 June 1993 concerning medical devices" are met. **Electromagnetic compatibility:** The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. You can find details on this measurement data on the separate enclosure.

Technical specifications

Name and model	: MEDISANA Blood pressure monitor BW 310
Display system	: Digital display
Memory slots	: 2 x 60 measurement values
Measuring method	: Oscillometric
Power supply	: 3 V _{max} ; 2 x 1.5V batteries AAA LR03
Blood pressure measuring range	: 30 – 250 mmHg
Pulse measuring range	: 40 – 180 beats/min.
Maximum error tolerance for static pressure	: ± 3 mmHg
Maximum error tolerance for pulse rate	: ± 5 % of the value
Pressure generation	: Automatic with pump
Deflation	: Automatic
Automatic switch-off	: After approx. 2 minutes
Operating conditions	: +5 °C to +40 °C, 15 to 93 % max. relative humidity; Air pressure 700 hPa to 1060 hPa
Storage conditions	: -25 °C to +70 °C, up to 93 % max. relative humidity; Air pressure 700 hPa to 1060 hPa
Dimensions (main unit)	: Approx. 72 x 75 x 30 mm (L x W x H)
Cuff	: 13.5 - 21.5 cm for adults
Weight	: Approx. 110 g without batteries
Item number	: 51070
EAN number	: 40 15588 51070 0

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **MEDISANA** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
 - All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - Accessories which are subject to normal wear and tear as batteries etc..
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana GmbH, Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY.
E-mail: info@medisana.de, Internet: www.medisana.com

For service, accessories and spare parts, please contact:

Murrays Health & Beauty
School Lane
Chandlers Ford
Hampshire
UK SO534YN
Tel. +44(2380)460600
eMail: cservices@paulmurrayplc.co.uk

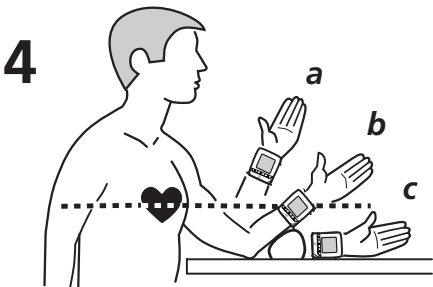
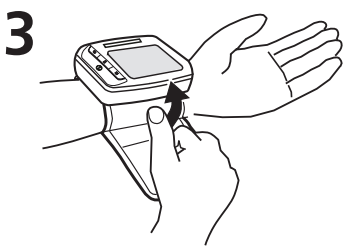
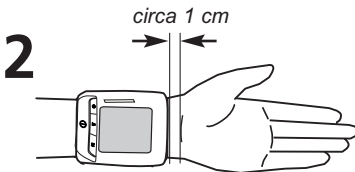
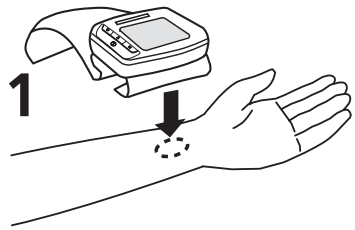
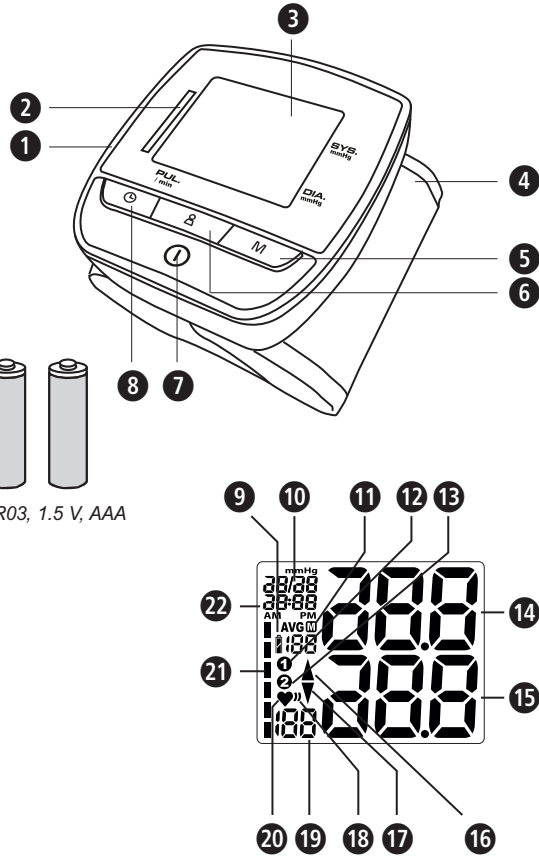
Device classification: type BF applied part

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