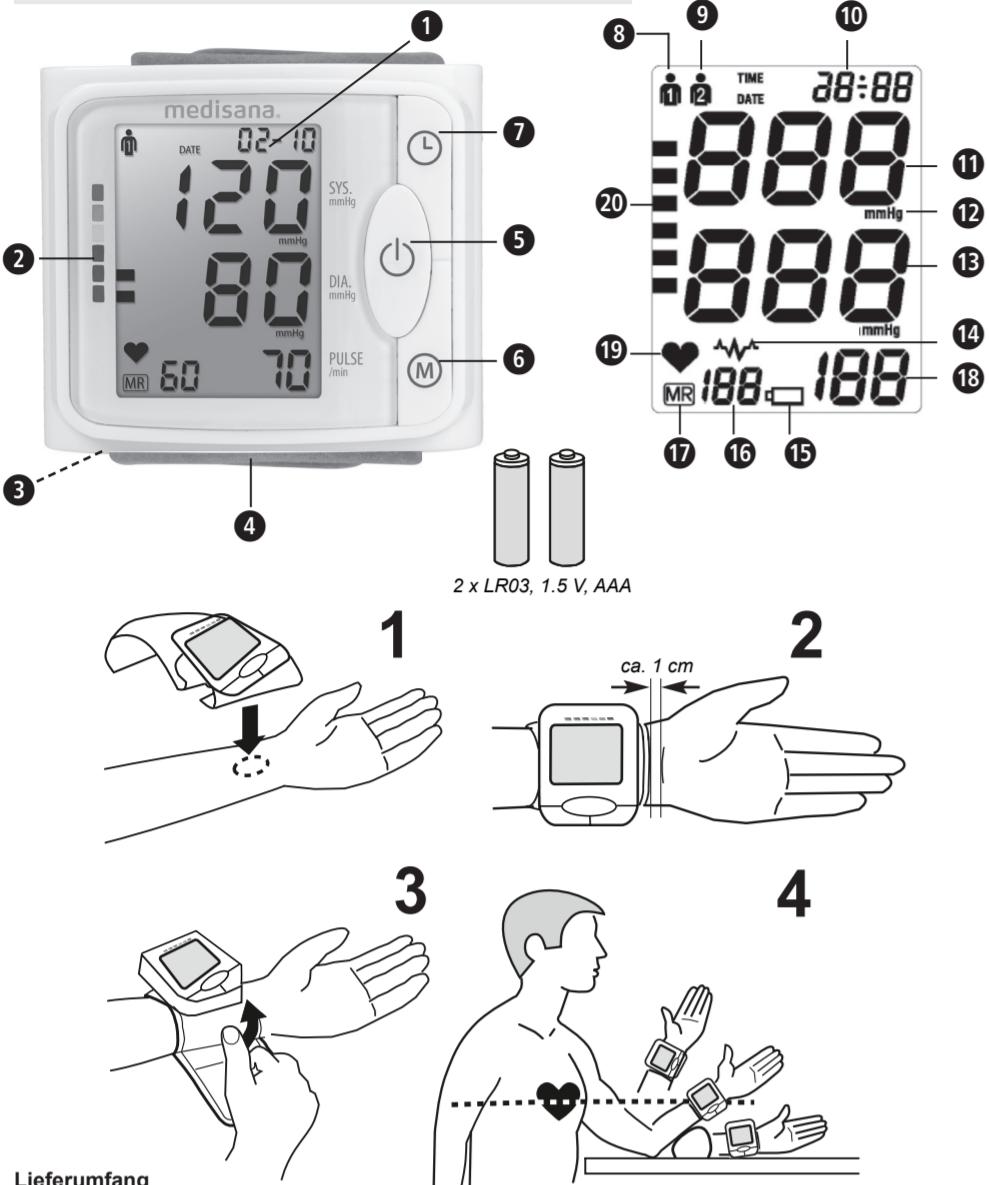
medisana

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
medisana

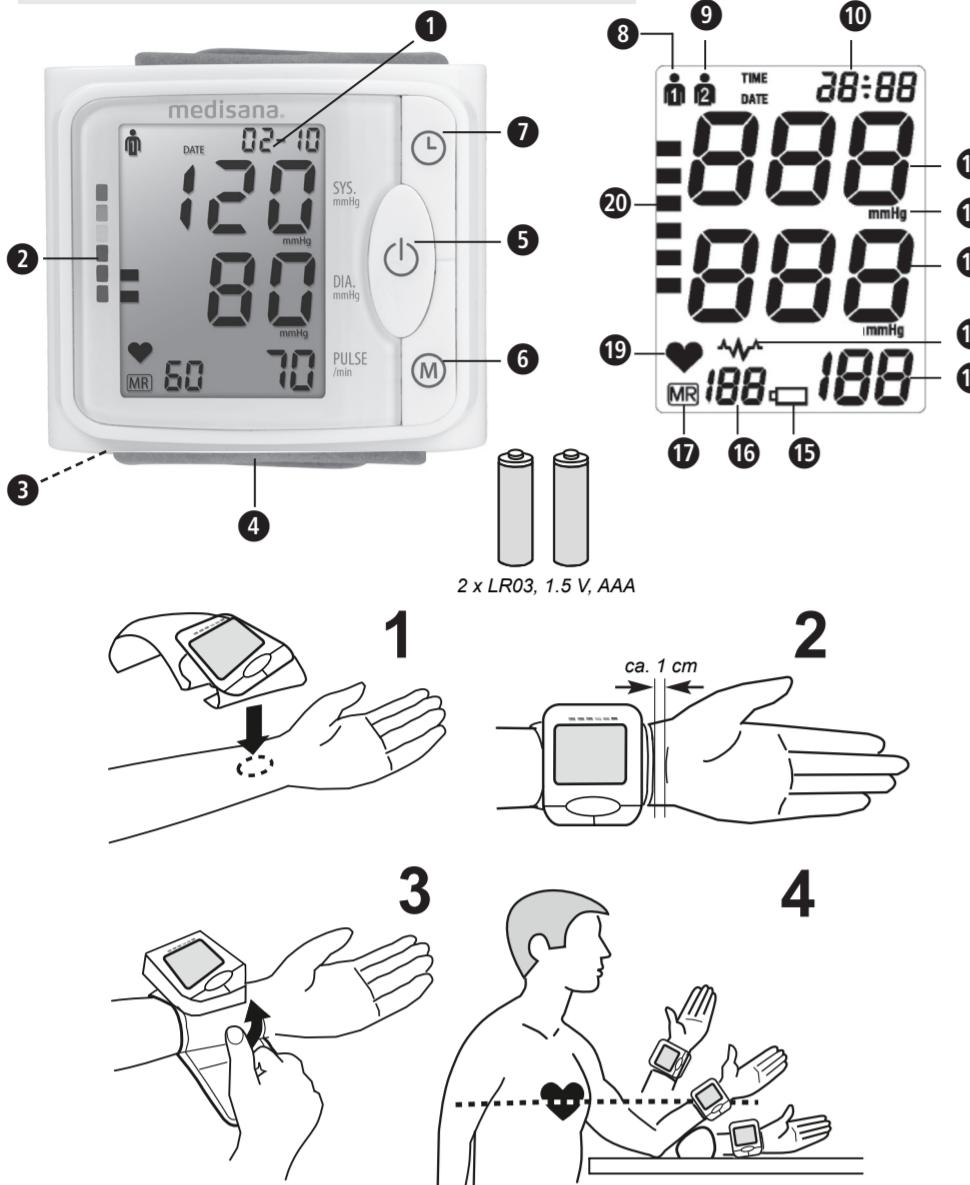GB Instruction manual Blood Pressure Monitor BW 320

Scope of supply
Please check first of all that the unit is complete. The following parts are included as standard:

- 1 medisana Blood Pressure Monitor BW 320
- 2 batteries (type AAA, LR03) 1.5 V
- 1 storage box
- 1 instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.

Explanation of symbols

This instruction manual belongs to this device.
It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Device classification:
type BF

SN Serial number

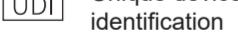
Protection class II


LOT LOT number

Manufacturer

MD Medical device

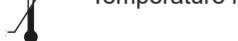
Importer

EC REP EU authorized representative

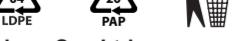Recycling symbols / codes:
These are used to provide information about the material and its proper use and recycling.

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- If you experience discomfort during a measurement, such as pain in the wrist or other complaints, press the **O**-button **⑩** to release the air immediately from the cuff. Loosen the cuff and remove it from your wrist.
- Blood pressure measurements in high repetition rates may lead to unwanted side effects, e.g. neural compression or blood clots.
- Blood pressure measurements - especially if executed frequently - may lead to temporary marks on the skin and / or to blood circulation disorders. If necessary, contact your physician to get more information.
- This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons.
- Do not allow children to use the unit. Medical products are not toys!
- Please ensure that the unit is kept away from the reach of children.
- The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.
- The user needs to check that the equipment functions properly and that it is in proper working condition before being used.
- The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves or together with HF surgical equipment, as this could cause functional defects or incorrect measurements.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

Blood pressure classification

systolic mmHg	diastolic mmHg		Bloodpressure indicator ②/⑩
≥ 180	≥ 110	severe hypertension	red
160 - 179	100 - 109	moderate hypertension	orange
140 - 159	90 - 99	mild hypertension	yellow
130 - 139	85 - 89	high-normal blood pressure	green
120 - 129	80 - 84	normal blood pressure	green
< 120	< 80	optimal blood pressure	green

WARNING

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell) the **O**-button **⑩** can be pressed at any time. The device immediately decreases the cuff pressure automatically.

Display stored values

This unit features 2 separate memories, each with a capacity of 120 memory slots. Results are automatically stored in the memory selected by the user. Now press the **M**-button **⑪** and the average values of the last 3 measurements appear on the display. Press the **M**-button **⑪** again and the latest stored measurement value will be shown, together with the memory symbol **MR*** **⑫** and the memory location place, **1*** **⑬**. Repeatedly pressing the **M**-button **⑪** displays the respective values measured previously (MR2, MR3 and so on). If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 3 min. Memory recall mode can be exited at any time by pressing the **O**-button **⑩**. The oldest value is deleted, if 120 measured values have been stored in the memory and a new value is saved.

Delete saved values

If you are sure that you want to permanently remove all stored memories of a user, press and hold the **M**-button **⑪** for approx. 5 seconds, until **CL** appears in the display. Release the button. Now press the **M**-button **⑪** again to confirm the deletion.

Error indicators

The following error codes will appear on the display when a measurement is abnormal:

Display	Cause
ERR1	No pulse was detected.
ERR2	Movement or talking during measurement.
ERR3	Inflation process lasts too long. The cuff probably has not been positioned properly.
ERR5	Systolic and diastolic pressure are too far apart from each other. Repeat the measurement. If you constantly receive unusual measurement results, please contact your doctor.

Starting up

Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment **①** is located on the left hand side of the unit. Open it and insert the 2 AAA LR03 type 1.5 V batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment. Replace the batteries immediately, if the change battery symbol **⑭** **⑮** appears on the display **①** or if nothing appears on the display after the unit has been switched on.

Setting date and clock

After insertion of the batteries you need to set the date and clock. To enter the setting mode, press and hold the **O**-button **⑩** for approx. 3 seconds. Firstly, the user memory 1 or 2 **⑯/⑰** is flashing. Now press the **O**-button **⑩** again to adjust the correct year figure. Adjust to correct year by pressing **M**-button **⑪** and **O**-button **⑩** to confirm the setting. The device now switches to the month setting mode. Proceed accordingly (in the same way as with the year adjustment) to adjust the correct figures for month, day, hour and minute. Afterwards, the setting procedure is complete. Shortly, the current date appears and then the clock appears. If you want to look at the date later on, press the **O**-button **⑩** shortly. If the batteries are changed, the set information is lost and needs to be entered again.

Setting up the user memory

The medisana wrist blood pressure monitor BW 320 gives you the opportunity to assign readings to one of two user memories, each of which provides 120 memory slots. To select the user memory, press and hold the **O**-button **⑩** for approx. 3 seconds. The user memory 1 or 2 **⑯/⑰** is flashing. Now press **M**-button **⑪** to select the user memory **⑯** or **⑰**. Confirm by pressing button **O**. Your selection will be saved by the device and will be considered in next blood pressure measurement respectively when the measurement result is saved. Your selection is valid until you make another selection.

Fitting the cuff

1. Attach the clean cuff to the left wrist which is free from clothing with the palm of the hand facing upwards (fig. 1).
2. There should be a gap of approx. 1 cm between the cuff and the palm of the hand (fig. 2).
3. Press the Velcro tape firmly around the wrist ensuring that it is not so tight that it could falsify the measured result (fig. 3).

Correct position for taking readings

- Carry out the measurement whilst seated.
- Relax your arm and place it comfortably, on a table for instance.
- Raise the wrist so that the cuff of the monitor is at heart level (fig. 4, position a = too high, b = correct, c = too low).
- Keep still during the measurement: Do not move and do not talk as this may alter the measurements.

Taking a blood pressure measurement

After the cuff has been appropriately positioned, the measurement can begin:

1. Press the **O**-button **⑩** to start the measurement.
2. The device automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display.
3. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol **⑯** **⑰** starts to flash on the display.
4. When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display **①**. The blood pressure indicator **⑩** is shown next to the relevant coloured bar depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the symbol **⑮** **⑯** **⑰** is also shown.

WARNING

Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

Technical specifications

Name and model	medisana Blood pressure monitor BW 320 (model: BP885W)
Display system	Digital display
Memory slots	: 2 x 120 measurement values
Measuring method	Oscillometric
Power supply	: 3V=, 2 x 1.5V batteries AAA LR03
Blood pressure measuring range	: 30 - 280 mmHg
Pulse measuring range	: 40 - 199 beats/min.
Maximum error tolerance for static pressure	: ± 3 mmHg
Maximum error tolerance for pulse rate	: ± 5 % of the value
Pressure generation	: Automatic with pump
Deflation	: Automatic
Automatic switch-off	: After approx. 3 minutes
Operating conditions	: +5 °C to +40 °C, 15 to 85 % max. relative humidity
Storage conditions	: -10 °C to +55 °C, 15 to 95 % max. relative humidity
Dimensions (main unit)	: approx. 75 x 64 x 29 mm (L x W x H)
Cuff	: 12.5 - 21.5 cm for adults
Weight	: approx. 117 g without batteries
Artikel-Nummer	: 51074
EAN-Nummer	: 40 15588 51074 8

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for medisana products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear as batteries etc.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

The service centre address is shown on the attached leaflet.

Common factors of wrong measurements

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