DE	Blutdruckmessgerät mit Radio MTR
GB	Bloodpressure monitor with radio MTR
FR	Tensiomètre avec radio MTR
IT	Misuratore di pressione con radio MTR
ES	Tensiómetro con radio MTR
PT	Aparelho de medição da pressão sanguínea com rádio MTR
NL	Bloeddruk meettoestel met radio MTR
FI	Radiolla varustettu verenpainemittari MTR
SE	Blodtrycksmätare med radio MTR
GR	Συσκευή μέτρησης της πίεσης με ραδιόφωνο MTR

CE 0297

Gebrauchsanweisung
Instruction Manual
Mode d'emploi
Istruzioni per l'uso
Instrucciones de manejo

Bitte sorgfältig lesen!
Please read carefully!
A lire attentivement s.v.p.!
Da leggere con attenzione!
¡Por favor lea con cuidado!

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GB

- ❶ Latch for storage case for upper arm cuff
- ❷ **RADIO** button
- ❸ **SET** button
- ❹ **“+/A”**-button
- ❺ **“-/B”**-button
- ❻ **START** button
- ❼ **MEM** button
- ❽ Set volume
- ❾ **ENTER** button
- ❿ Storage case for upper arm cuff
- ⓫ Display
- ⓫ Operation mode indicator: Blood pressure
measurement
- ⓫ Operation mode indicator: Memory
- ⓫ Operation mode indicator: Settings
- ⓫ Operation mode indicator: Radio
- ⓫ Socket for mains adaptor
- ⓫ Battery compartment
- ⓫ Blood pressure indicator
- ⓫ Batteries low
- ⓫ **Voice** symbol
- ⓫ **Radio clock** symbol
- ⓫ **Alarm** symbol
- ⓫ User memory 1 / 2
- ⓫ Display of mode, temperature, pulse or
radio frequency (depending on
operation mode)
- ⓫ Pulse symbol (arrhythmia display)
- ⓫ Display of humidity or memory location
number (depending on operation mode)
- ⓫ **M** symbol (Memory)
- ⓫ Display of date/clock or alarm time
(depending on operation mode)
- ⓫ Display of clock resp. year or systolic
pressure/diastolic pressure or language
number (depending on operation mode)

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols:

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING!

These warning notes must be observed to prevent any injury to the user.

CAUTION!

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Classification: IIa, type B

LOT number

Manufacturer

Date of manufacture

Safety Information

1.1

Tips for your well being

**WARNING**

Do not take any therapeutic measures based on your own measurements! Never change the quantity of medication prescribed by your doctor!

- Irregularity of pulse or arrhythmia can lead to difficulties in recording a correct reading when measurements are taken using oscillometric blood pressure devices.
- If you should suffer from conditions such as arterial occlusive disease, please consult your doctor before using the unit.
- The unit may not be used to check the heart rate of patients with a pacemaker.
- Pregnant patients should take their own personal state of health into consideration and take all necessary precautions before using the unit. If in doubt, consult your doctor.
- Please take the following measures if anything untoward should happen such as pain in the upper arm or other discomfort: Press the **START** button **6** to deflate the cuff immediately. Loosen the cuff and remove it from your arm.
- Patients with a weak pulse wave should raise their arm and open and close their hand around 10 times before taking a reading. This exercise optimizes the pulse wave and the measuring process.
- The unit is only to be used for the specific purpose described in this instruction manual.
- Any misuse will void the warranty.
- This unit is intended for domestic use only.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not use the unit when it does not work correctly or if it has dropped or fallen into water or when it was damaged in any other way.
- On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately.
- Do not use the unit near to equipment emitting a strong electric or magnetic field, such as radios. This may impair the correct functioning of the unit.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.

1.2

Important considerations

- If a fault occurs, do not try to repair the unit yourself. Attempts to do so will void the warranty. Refer all servicing to authorized service personnel.
- Protect the unit from moisture.
- Never touch an appliance that has fallen into water. Unplug the unit from the mains outlet immediately.
- You must not continue to use the unit if the lead or the mains adaptor are damaged. For safety reasons, these parts may only be replaced by authorized service personnel.
- You must never submerge the unit, the lead or the mains adaptor in water or hold them under running water.
- Do not store the lead near a heat source or drag it over sharp edges.
- Only handle the mains adaptor when your hands are dry.
- Never carry, pull or turn the unit by the mains lead and do not let the lead get tangled.
- Protect the unit from shocks.
- Place the appliance on flat and even surface, so that it cannot tip over or fall down.

2 Useful Information

Thank you for your confidence in us and congratulations on your purchase!

With the **Bloodpressure monitor with radio MTR** you have purchased a **MEDISANA** quality product. To ensure the best results and long-term satisfaction with your **MEDISANA Bloodpressure monitor with radio MTR**, we recommend that you read the following operating and maintenance instructions carefully.

2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. If in doubt, do not use it. Send it to a service point.

The following parts are included:

- 1 **MEDISANA Bloodpressure monitor with radio MTR**
- 1 Cuff with air hose
- 1 mains adaptor with connection cable
- 4 batteries 1.5V (type AA, LR6)
- 1 instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

2.2 What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

2.3 How is blood pressure measured?

The **Bloodpressure monitor with radio MTR** is a blood pressure monitor intended for measuring blood pressure on the wrist. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures fluctuations in pressure resulting from the inflation and deflation of the cuff over the artery.

2.4 Why should you take your blood pressure at home?

MEDISANA has had many years of experience in the field of blood pressure measurement. The high precision of **MEDISANA** devices has been confirmed by extensive clinical tests, carried out according to strict international standards. One strong argument in favour of taking blood pressure at home is the fact that the readings are taken in familiar surroundings under relaxed conditions. The most important reading is the so-called „baseline value“, the one taken before breakfast, directly after getting up in the morning.

As far as it is possible, you should always try to take your blood pressure at the same time and under the same conditions each day. This ensures comparable results and helps in detecting the first signs of high blood pressure. If high blood pressure remains undetected, it increases the risk of further cardiovascular diseases.

THEREFORE WE RECOMMEND:

Take a blood pressure reading daily and on a regular basis, even if you are experiencing no apparent complaints.

2.5 Blood pressure classification

The following table shows the classification of high and low blood pressure, without taking age into consideration. This blood pressure assessment scale corresponds to the guidelines of the World Health Organisation (WHO).

Low blood pressure:

systolic below <100 diastolic below <60

Normal blood pressure (green display area 18):

systolic 100 - 139 diastolic 60 - 89

Types of high blood pressure:

Mild hypertension (yellow display area 18):

systolic 140 – 159 diastolic 90 – 99

Medium hypertension (orange display area 18):

systolic 160 – 179 diastolic 100 – 109

Serious hypertension (red display area 18):

systolic ≥ 180 diastolic ≥ 110

WARNING

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

2.6 Fluctuations in blood pressure

There are many factors that can influence blood pressure. Readings can be affected seriously by stress, fear, heavy physical exertion or the time of day at which the measurements are taken. Your personal blood pressure values are subject to considerable fluctuation during the course of the day and the year. These fluctuations are particularly pronounced in the case of patients suffering from hypertension. Blood pressure is usually at its highest during physical exertion and at its lowest at night when you are asleep.

2.7 Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

3 Getting started

3.1 Set-up and installation

1. Place the unit on a flat and even surface, so that it cannot tip over or fall down.
2. If used as clock, radio or alarm clock place the unit in a distance of approx. 40 to 50cm away from your head e.g. on a table.
3. Connect the unit to a power supply. You may use the mains adaptor or the included batteries. If you use the mains adaptor, the display of time and date is of better quality than during the use of batteries. If batteries are inserted and the mains adaptor is connected to a power source, the unit will prefer the power supply by mains adaptor (batteries will be deactivated).

3.2 Inserting / removing batteries

Insertion: The lid of the battery compartment ⑪ is located on the bottom side of the unit. Open it and insert the 4 included batteries 1.5V, AA, LR6. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment ⑪. **Removal:** Replace the batteries when the batteries low symbol ⑫ appears in the display ("⚡"). If nothing is displayed the batteries are completely empty and need to be replaced immediately.

WARNING BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately! Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

3.3 Using a mains adaptor

Alternatively, the unit may also be operated with the included power adaptor (6V, 800mA), which is plugged into the connector ⑬ provided on the bottom side of the unit. In this case the batteries remain in the unit. The batteries are disconnected mechanically when the connector is inserted.

3.4 Setting time and date

Your **MEDISANA Bloodpressure monitor with radio MTR** is equipped with a DCF77 RCC (Radio Controlled Clock). The DCF77-signal automatically adjusts time and date. If there is no DCF77-signal, time and date will not be set automatically.

Automatic time/date-setting:

As soon as you have connected the item to a power source, the device automatically switches to the radio clock mode. While searching for the time signal (DCF77-signal), the **radio clock** ⑭ flashes. As soon as the clock receives an adequate signal for setting, the time and date are set automatically. Pay attention to a good receiving position and avoid radio screening by metallic items or building parts. You may change the position of the unit to improve the receipt of the DCF77signal.

NOTE

If you want to leave the automatic time setting mode, press the **START** button ⑥.

The automatic adjustment of time and date may last up to 9 minutes. If no signal is received within this time, time and date will continue to run without change (e.g. 0:00.....0:01.....0:02 etc.). You may now set time and date manually. If you have changed the position of the unit, a new DCF77-signal receiving process may be started by pressing **SET** button ③, selecting the radio clock symbol (the symbol will flash) with the buttons “+/A” ④ resp. “-/B” ⑤ and another press on the **SET** button ③. If no receipt is possible, the unit will automatically switch to the manual time and date setting mode.

NOTE

The device automatically receives the radio signal to update the time setting twice a day, at 2.05 a.m. and 2.05 p.m. During this process, the Radio clock symbol ②① is flashing. By pressing the **START** button ⑥ the regular reception of the time signal can be switched off.

Manual adjustment:

Press the **SET** button ③ to enter the setting mode. The LED for this mode ⑭ is flashing. Select the time-/date-display (display will flash) with the buttons “+/A” ④ resp. “-/B” ⑤ and press **SET** button ③ again to get into manual adjustment mode. At first, the year figure will flash. Adjust the value with the buttons “+/A” ④ resp. “-/B” ⑤ and confirm with **SET** button ③. Repeat this process in the same manner for month, day, hour, minute and time difference (you can adjust the time difference of the actual time zone in regard to the middle european time zone MEZ). After finishing all inputs, press **SET** button ③ once again.

NOTE

If you want to leave the manual adjustment mode, press **START** button ⑥. The unit automatically leaves the mode after approx. 1 minute without any inputs.

3.5 Adjusting the alarm

Press the **SET** button ③ to enter the setting mode. The LED for this mode ⑭ is flashing. Select the **Alarm**-symbol ②② (symbol will flash) with the buttons “+/A” ④ resp. “-/B” ⑤ and press **SET** button ③ again to get into alarm adjustment mode. Select one of the following main alarm functions: “OnA” = wake-up with alarm tone; “OnS” = wake up with radio or “Off” = alarm deactivated. Select the desired function with the buttons “+/A” ④ resp. “-/B” ⑤ and press **SET** button ③ again.

If you have deactivated the alarm, you will now return to the display of time and date. Is the alarm activated, you can now enter the hour and minute of the alarm time by using the button “+/A” ④ resp. “-/B” ⑤ and **SET** button ③ to confirm. After the adjustment of the minutes and pressing **SET** button ③ you will return to the display of time and date. With the Set volume button ⑧ you can adjust the volume of the alarm. Upon reaching the alarm time, the alarm will beep for 1 minute. Press any button to end the alarm.

NOTE

If you want to leave the alarm adjustment mode, press **START** button ⑥. The unit automatically leaves the mode after approx. 1 minute without any inputs.

3.6

Adjusting the voice function

The blood pressure monitor with radio MTR is equipped with a voice function, which can announce the date and time during its display. Press **ENTER** button ⑨ to activate this function. Furthermore, blood pressure measurement results can be announced with this voice function, please see chapter “Measuring the blood pressure”. The voice function can be adjusted in 7 different languages. For selecting a language (resp. for deactivating the voice function) press **SET** button ③ and select the voice symbol ⑳ (the symbol flashes) with the buttons “+/A” ④ resp. “-/B” ⑤ and press **SET** button ③ to confirm. You may now choose between these voice options:

L0 = Deactivate voice function

L1 = Activate voice function, Language: German

L2 = Activate voice function, Language: English

L3 = Activate voice function, Language: Spanish

L4 = Activate voice function, Language: Dutch

L5 = Activate voice function, Language: French

L6 = Activate voice function, Language: Italian

L7 = Activate voice function, Language: Portuguese

Select the desired option with the buttons “+/A” ④ resp. “-/B” ⑤ and press **SET** button ③ to confirm. For the options **L1** to **L7** there is a spoken announcement of the selected language. If the voice function has been deactivated (**L0** is shown in display) three beeps can be heard and the **voice** symbol ⑳ will extinguish. You may adjust the volume of the spoken words at any time with the set volume button ⑧.

NOTE

If you want to leave the voice adjustment mode, press **START** button ⑥. The unit automatically leaves the mode after approx. 1 minute without any inputs.

3.7

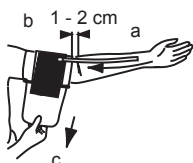
Radio function

If you want to hear Radio, press the **Radio** ② button. The operation mode indicator: Radio ⑮ will light up and the frequency ⑳, date ㉔ and time ㉚ will be displayed. With the buttons “+/A” ④ resp. “-/B” ⑤ you can start a scan for available radio stations. The volume can be adjusted with set volume button ⑧ To deactivate the radio function, simply press the **START** button ⑥.

4 Operating

4.1

Fitting the cuff

1. Open the storage case ⑩ by pressing the latch for storage case ①. Push the end piece of the air tube into the hole on the unit prior to use. Ensure that the hose is not damaged and is securely attached. The blood pressure monitor is intended for use as a table-top unit.

2. Slide the open end of the cuff through the metal bracket so that the Velcro fastener is on the outside.
3. Open the cuff so as to create a cylindrical shape.
4. Slide the cuff over your left upper arm. Position the air hose in the middle of your arm in line with your middle finger (a).
5. Place your arm on the table with the palm uppermost. The lower edge of the cuff should be 1 - 2cm above the crease of the elbow (b). Pull the cuff tight (c) and close the Velcro fastener.
6. Avoid creating a space between arm and cuff. It may falsify the measured result.
7. Measure the pulse on your bare arm. If wearing tight clothing, ensure that the circulation in the arm is not impaired when rolling up the sleeve.
8. Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.

4.2

Correct position for taking readings

The Correct Measuring Position when sitting

- Carry out the measurement whilst seated.
- Relax your arm and place it comfortably, on a table for instance.
- Ensure that the cuff is at heart height.
- Keep still during the measurement: do not move and do not talk as this may alter the measurements.

Correct position for taking measurements lying down

- In special cases the measurement can also be taken lying down.
- Lie on your back and relax for at least 5 minutes and also relax your arm.
- Ensure that the cuff is at heart height. Differing heights affect the accuracy of the measurement.
- Keep still during the measurement: do not move and do not talk as this may alter the measurements.

4.3

Taking blood pressure readings

1. Press “+/A” **4** resp. “-/B” **5** to select memory 1 or 2 **23** (the corresponding symbol appears: **1 2**) or the guest function (no symbol will be displayed).
2. Now press the **START** button **6**. A beep is heard and all display characters are shown (display test). After approx. 1 second the display shows the values from the previous measurement. The operation mode indicator: Blood pressure measurement **12** appears.
3. If the voice function is activated, important hints will be given.
4. The unit is ready for measurement and automatically inflates the cuff slowly in order to measure your blood pressure. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. It is possible, that the unit pumps more air into the cuff, when the pressure is insufficient. During the measurement the blood pressure indicator **18** is constantly changing.
5. When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure **29** and the pulse value appear on the display. The blood pressure indicator **18** flashes next to the relevant coloured bar depending on the WHO blood pressure classification. If the unit has detected pulse arrhythmia, the **pulse** symbol **25** flashes.
6. Is the voice function activated, all measured values will additionally be announced.
7. If a memory (1 or 2) has been selected in step 1, the measured values will automatically be stored. Up to 60 measured values with date and time may be stored in the memory.

NOTE

If you want to abort the measurement manually, press the **START** button **6**. The cuff will be deflated, thereafter it can be removed from the arm.

4.4

Switching off the unit

The unit has an automatic switch-off function. The unit switches off automatically after a minute if no buttons are pressed. The unit may also be switched off using the **START** button **6**. The time and the date are constantly shown in the display.

5.1 Displaying stored results

Up to 60 measurements (blood pressure and pulse with time and date as well as blood pressure indicator 18) may be stored in each of the blood pressure monitor's two memory slots. To call up the automatically stored values, first choose the memory slot (1 or 2) 23 with the buttons "+/A" 4 resp. "-/B" 5. The corresponding symbol will appear: 1 2. Now press the **MEM** button 7. A beep can be heard and the operation mode indicator: Memory 13 lights up. First, the number of stored values for this user is displayed. After approx. one second an "A" appears next to the **M-Symbol** 27 and the average values for systolic and diastolic pressure and pulse out of all stored data for this user are displayed. Is the voice function activated, these values will also be announced in the selected language. Are no values stored for this user, a "0" appears. Repeatedly pressing the **MEM** button 7 displays the respective average values out of all stored data within the last 7 days of this user. Is the voice function activated, these values will also be announced in the selected language. "A7" is displayed next to **M-Symbol** 27. Another press on the **MEM** button 7 displays the measured values for systolic and diastolic pressure, pulse and blood pressure indicator 18 of the most recent measurement. Is the voice function activated, these values will also be announced in the selected language. You may use the buttons "+/A" 4 resp. "-/B" 5 to recall previously stored values. Together with the stored values the **M-Symbol** 27 and next to it the memory location number 26 are shown. Press the **START** 6 button to leave this mode at any time and return to the time & date display. If not button is pressed for more than approx. 2 minutes, the unit automatically returns to the time & date display. If all 60 memory locations in a memory are occupied, a new measured value is saved whilst the oldest value is deleted.

5.2 Deleting memory

During the display of stored values (however NOT during the display of average values "A" or "A7") you can delete the stored values of the selected user memory slot (1 or 2). Press and hold the **MEM** button 7 for at least 3 seconds to delete the values. 3 beeps can be heard as confirmation for the deletion. Press **MEM** 7 button again or press **START** button 6 to return to the display of time & date.

6 Miscellaneous

6.1 Troubleshooting

The batteries low symbol 19 " " appears in the display:

- The batteries are too low or empty. Replace all four batteries with new 1.5V alkali AA batteries, LR6.

This unit detects different causes of errors and displays different error codes (depending on the cause). An eventual measurement will be interrupted.

“Er 0”: Pressure system is unstable before measurement. Fit cuff correctly and tighten and repeat the measurement according to the instructions for use.

“Er 1”: Fail to detect systolic pressure. Please repeat the measurement according to the instructions for use.

“Er 2”: Fail to detect diastolic pressure. Please repeat the measurement according to the instructions for use.

“Er 3”: Pneumatic system blocked or cuff is too tight during inflation. Fit cuff correctly and tighten and repeat the measurement according to the instructions for use.

“Er 4”: Pneumatic system leakage or cuff is too loose during inflation. Fit cuff correctly and tighten and repeat the measurement according to the instructions for use.

“Er 5”: The pressure in the cuff is too high (over 300 mmHg) or the pulse (over 200 beats per minute) is too high. Relax for 5 minutes and repeat the measurement.

“Er 6”: Inflation takes too long. Fit cuff correctly and tighten and repeat the measurement.

“Er 7”: There is an electronical problem with the unit. Remove the batteries and disconnect the adaptor cable. Wait for 5 minutes and re-connect the power supply. Repeat the measurement according to the instructions for use.

“Er 8”: Device parameter checking error. Relax for 5 minutes and repeat the measurement according to the instructions for use.

“Er A”: Pressure sensor parameter error. Relax for 5 minutes and repeat the measurement according to the instructions for use.

Important: Please contact the service centre if the error message appears again when the measurement is repeated. Keep still and do not move your arm or the cuff during measurement. The unit is not suitable for people with an extremely irregular heartbeat.

6.2

Care and maintenance

- Remove the batteries before cleaning the unit.
- Clean the unit using a soft cloth lightly moistened with a mild soapy solution.
- Never use abrasive cleaning agents or alcohol.
- Do not expose the unit to direct sunlight and protect it from dust and moisture.

6.3

Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

6.4

Directives / norms

This blood pressure monitor with radio MTR meets the requirements of the EU standard for non invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE 0297". The blood pressure monitor corresponds to European standards EN 60601-1, EN 60601-1-2, EN 1060, EN 14971 and EN 980. The specifications of EU Guideline "93/42/EEC of the Council Directive dated 14 June 1993 concerning medical devices" are met.

Electromagnetic compatibility:

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Inquire at **MEDISANA** for details on this measurement data.

Intended use:

This unit is suitable for non-invasive blood pressure measurement in adults (i.e., suitable for external use).

6.5

Technical specifications

Name and model:	MEDISANA blood pressure monitor with radio MTR
Display system:	Digital display
Memory slots:	2 x 60 incl. time and date
Measuring technique:	Oscillometric
Power supply:	230V~50Hz. (adaptor 6V, 800mA) or 6V, 4 x 1.5V batteries AA / LR6
Blood pressure measuring range:	30 - 250 mmHg
Pulse measuring range:	40 - 180 beats / Min.
Maximum error tolerance for static pressure:	+/- 3 mmHg
Maximum error tolerance for pulse rate:	+/- 5% of the reading
Pressure generation:	Automatic, using pump
Deflation:	Automatic
Radio frequency range:	87 - 108 Mhz
Operating conditions:	+5°C to +40°C; < 80% humidity
Storage conditions:	-20°C to +55°C; < 95% humidity
Dimensions:	Approx. 203 x 180 x 81 mm
Cuff size:	Circumference 22 -30 cm Adult cuff
Weight approx.:	802 g incl. batteries & cuff
Article number:	51097
EAN number:	40 15588 51097 7

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (e.g. cuff, batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

 MEDISANA AG

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41468 NEUSS

Germany

eMail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the attached leaflet.

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