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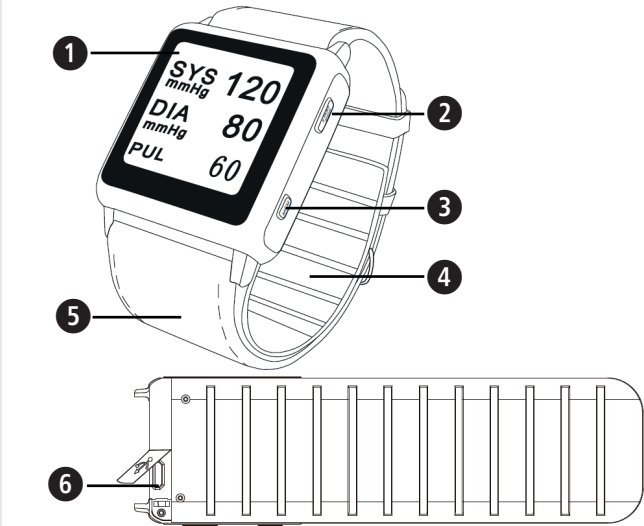
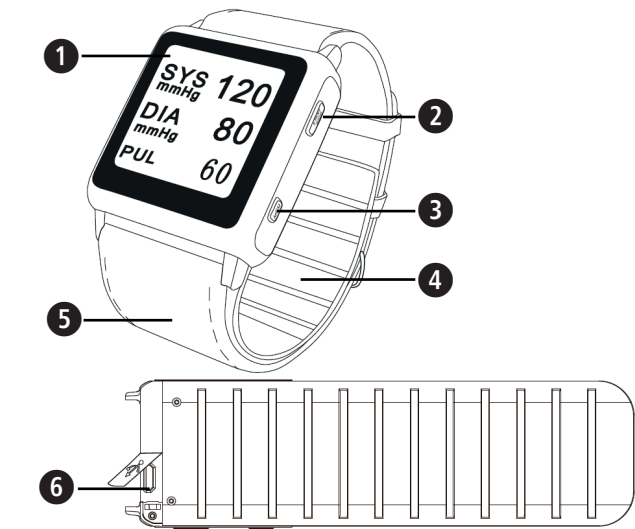
Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

IP22 Ingress protection rating

Type BF applied part

LOT Lot number

Manufacturer

Date of manufacture

Keep dry

EC REP Authorized representative in the European Community

SN Specifies serial number of the device

GB IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Safety notes

- Contact your physician if you suspect you have a medical issue.
- Not for use by children, pregnant women, or pre-eclamptic patients.
- If you suffer from arrhythmia, diabetes, blood circulation problems or apoplexy, only use the device under a physician's care.
- Prolonged over-inflation of the monitor will result in harmful injury to the patient.
- Do not use this device over a wound, as this can cause further injury.
- Too frequent measurements can cause injury due to blood flow interference.
- This device possibly cause improper result due to external interference, such as acceleration during transport or transport in general.
- Do not attempt to modify this device in any way.
- Store this device in a cool, dry place. Do not subject this device to extreme temperatures, humidity or sunlight. This device might not meet performance specifications if stored or used outside the ranges specified in the "Specifications" section. Keep away from rain.
- Do not immerse this device in water or clean with cleaning products, alcohol or solvents. Carefully follow cleaning instructions provided.
- Ensure that the device is used in the environment specified in the EMC declaration in this instruction manual, otherwise, may result in improper operation.
- Use of this device adjacent to or stacked with other device should be avoided because it could result in improper operation.
- This electrical medical equipment requires specific precautions regarding electromagnetic compatibility. It must be installed and used according to the electromagnetic information.
- Do not use this device where flammable gases or liquids are present.
- Never drop the device.
- Do not replace the wristband with the one not from the manufacturer, or may result in inaccurate measurement result. The cuff fits wrist circumference ranging from 5.3 inches to 8.5 inches (approx. 13.5 cm to 21.5 cm), the user should confirm if the cuff fits the wrist before purchasing/using the device.

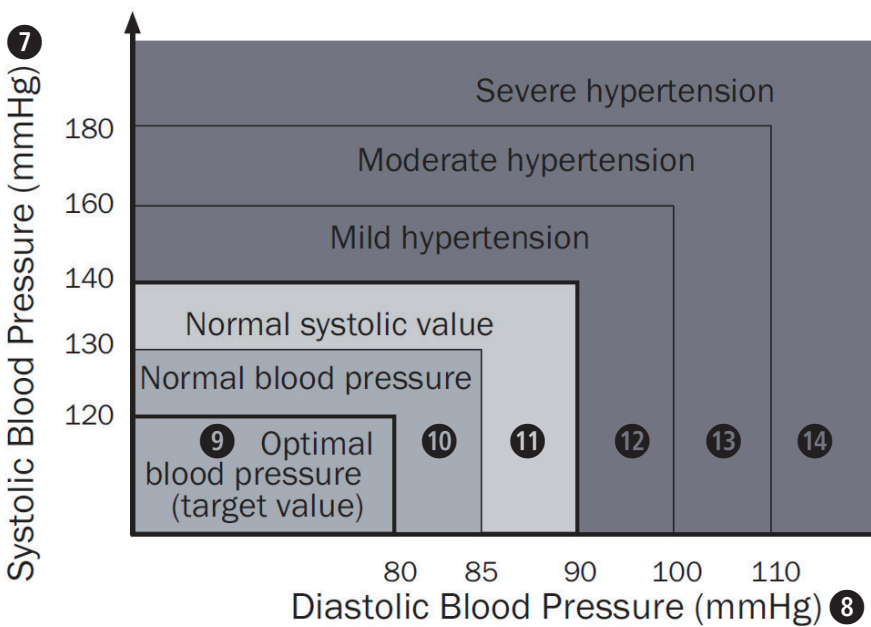
Calibration and Service

The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life. It is generally recommended to have the device inspected and calibrated every two years to ensure correct functioning and accuracy. Please consult your authorized distributor.

Intended use

This wrist blood pressure monitor is intended to be used to measure blood pressure (systolic and diastolic) and heart rate from the wrist by using the oscillometric method. The device is intended for using in only adult population, not applied to the other populations such as neonatal baby. It can't be used while the wrist (arm) has bleeding or wound to avoid the blood flowing from the wound in pressurizing. The cuff circumference is limited to 13.5 cm - 21.5 cm.

Blood pressure indicator chart

When systolic and diastolic pressures fall into different categories, the higher pressure determines color code. Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by European Society of Hypertension International Protocol revision 2010 for the validation of blood pressure measuring devices in adults.

- 7 Systolic blood pressure (mmHg)
- 8 Diastolic blood pressure (mmHg)
- 9 Optimal blood pressure (target value)
- 10 Normal blood pressure
- 11 Normal systolic value
- 12 Mild hypertension
- 13 Moderate hypertension
- 14 Severe hypertension

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre. The following parts are included:

- 1 medisana Blood pressure watch **BPW 300**

- 1 Instruction manual
- 1 USB cable
- 1 Soft ruler

WARNING
Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

Application

Before Taking a Measurement

- For most accurate results, measurements should be taken in the same circumstances and at the same time every day.
- The device can be applied to either wrist, but preferably the left.
- Use the same wrist for each measurement.
- Any blood pressure recording can be affected by the position and physiological condition of the subject, as well as other factors such as background noise, stress, talking and body movement etc.
- Avoid taking measurements while under stress.
- Do not eat, drink alcohol, smoke, exercise, or bathe 30 minutes before using the device, as these activities may change blood pressure.
- For an accurate indication of true blood pressure, take and record several measurements over a period of time.
- If consecutive measurements are desired, allow two minutes between uses to allow blood to flow through the arm normally.
- This device is used to monitor blood pressure. However, it cannot treat or prevent high blood pressure. If you suffer from high blood pressure, consult your doctor.

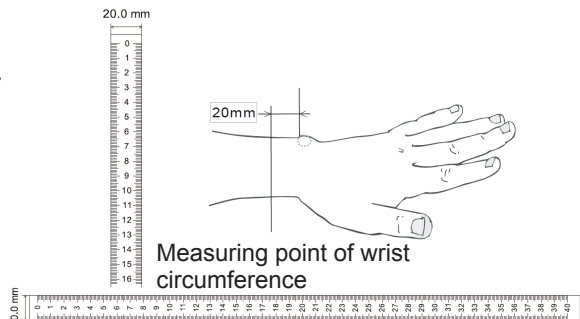
Correct wearing and measuring precaution

- Remove sleeve, wristwatch and other jewelry. The device should be used on bare skin. Loosely roll up your sleeve to expose wrist if necessary.

Note: Tightening by your sleeve may result in inaccurate measurement result.

Measure wrist circumference

- Measure wrist circumference by using a soft ruler at the measuring point which is 20mm away from root of styloid process of ulna as the figure on the right:
- Note: Inaccurate measuring wrist circumference may result in inaccurate measurement results.


Determine the hole number by which the wristband will be locked

- Determine the number of hole by which the wristband will be locked according to the wrist circumference measured above.


wrist circumference (mm)	135-140	141-147	148-157	158-167	168-178	179-184	185-192	193-200	201-208	209-215
hole number	H4	H5	H6	H7	H8	H9	H10	H11	H12	H13

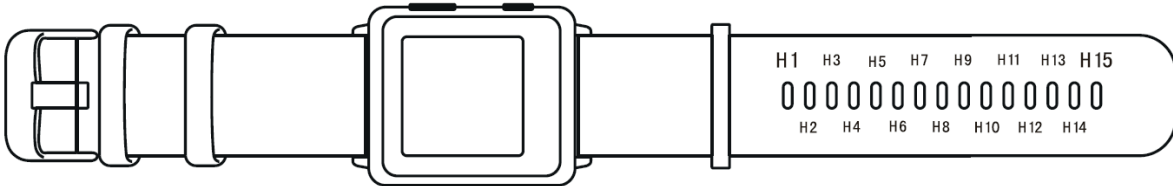Note: The incorrect number of hole for lock may result in inaccurate measurement results.

Wear the device

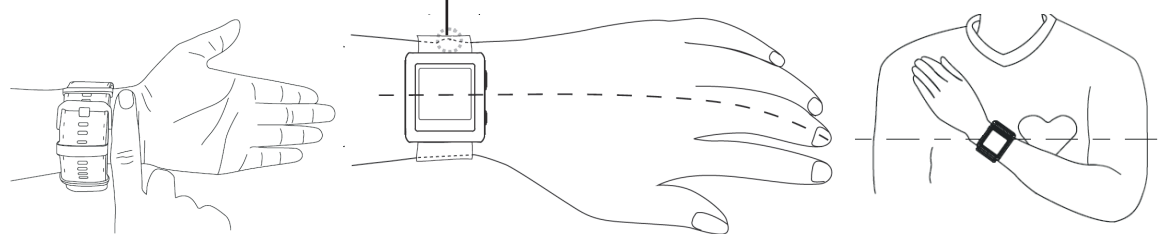Position the device and leave width of your forefinger (approx. 1.0 - 1.5 cm) between the wristband and the bottom of your palm. wrap and fasten wristband with cuff around your wrist properly.

Take a Measurement

- Comfortably seated • Legs uncrossed • Feet flat on the floor • Back and arm supported
 - Keep the device at the same level as your heart
 - Relax as much as possible and not talk during the measurement.
- While sitting upright in a chair with your feet flat on the floor, press START/STOP button 2. Cuff will begin to inflate. Do not move, activate arm muscles, or talk during measurement. Measurement can be stopped at anytime by pressing START/STOP button 2. When the device senses a pulse, a heart symbol will flash on screen. When measurement is complete, the cuff deflates automatically and the systolic pressure, diastolic pressure, and pulse rate will display on screen. MEM button 3 : press the button to recall last measurement.

Connecting to VitaDock+

Free download of the VitaDock+ app in the app store or Google Play store. Activate Bluetooth® on your iOS or Android appliance. Start the app. Touch the Bluetooth® symbol to open the list of devices. Scroll down the device list until you see the BPW 300. Touch the arrow beside the device to start the setup process. The display/synchronisation of the time is only possible with a connection to the VitaDock+ app.

Cleaning and maintenance

- For regular maintenance, the device only needs to be wiped gently with a soft, dry cloth while it is not being used. Never immerse the device or any components in water.
- Do not carry out repairs of any kind yourself. If a defect occurs, please contact your local authorized distributor. Use only authorized parts and accessories.
- The device's battery should be recharged periodically, even during storage since the rechargeable battery installed in the device will self-discharge, leading to an unacceptably low voltage, thus damaging the battery.

Troubleshooting

Screen text	Possible cause	Solution
Wearing loose	wearing is too loose	Check if the cuff is leaking. Maintenance may be needed if leaking. or: Wrap the cuff properly.
Body motion interference	body motion	Do not move your arm and body, keep quiet
Wearing tight	wearing is too tight	Wrap the cuff properly
Wearing inappropriate	wearing is not appropriate	Wrap the cuff properly
C6:D5:8D:94:ED:4A	Bluetooth code for binding	Please bind the collateral device and the BPW 300
	The battery voltage is too low	Please charge in time
	Charging mode	Device is charging
Wrong values	Change measurement unit	Press and hold button 2 and button 3 for 5 sec., then change unit with button 3 and confirm with button 2

Technical specifications

Name and model	medisana blood pressure watch BPW 300
Display	E-Ink LCD
Measuring Principle	Oscillometric Method
Measurable Circumference of Wrist	13.5~21.5cm
Electric Shock Protection	Internal power supply appliance type BF
Measurement Range	0 to 299 mmHg (0 to 39.9kPa); 40 to 180 Pulses/min
(Static pressure; Pulse)	
Accuracy	±3 mmHg (±0.4 kPa); ±5% of reading
(Static pressure; Pulse)	
LCD Indication (Pressure; Pulse)	Yes; Yes
Inflation	Automatic by Internal Air Pump
Rapid Air Release	Automatic by Air Valve
Power Source	Built-in 3.7V DC, 210mAh Li-ion Polymer Battery which should be charged by a charger of 5V DC, 500mA or more
USB	USB 2.0 Port only for battery charge
Bluetooth®	Bluetooth 4.0(data transmission), Frequency range:2.4GHz(2402 – 2480MHz),Modulation: GFSK, Effective radiated power: <20dBm within 2 minutes
Automatic turn off	approx. 65 g
Weight	2 Years
Lifespan	10 °C to 40 °C, 15 to 90 % relative humidity, non condensing, atmospheric pressure: 80-106 kPa
Operating conditions	- 20 °C to + 55 °C, 0 to 90 % relative humidity, non condensing
Storage and transport conditions	51215
Article No.	40 15588 51215 5
EAN number	

The device complies with IEC 60601-1:2005+A1:2012, IEC60601-1-2:2014, IEC60601-1-11:2015, IEC80601-2-30:2009+A1:2013, EN 1060-3: 1995+A2:2009

Disposal
This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.
The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms


- Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:
1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
 2. Defects in material or workmanship will be removed free of charge within the warranty period.
 3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
 4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
 5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

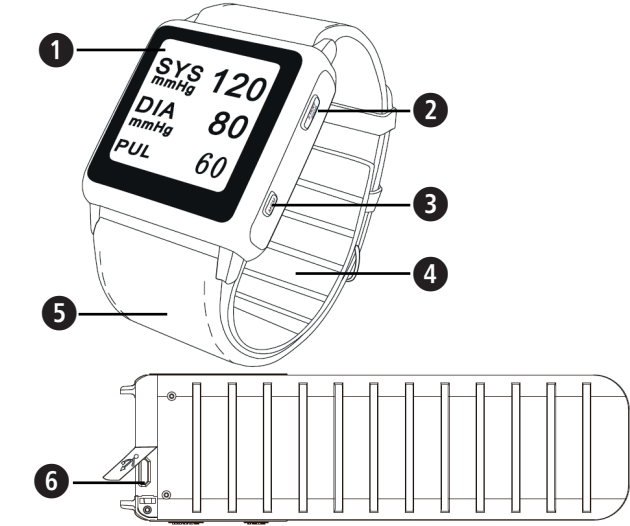Shenzhen Kingyield Technology Co.,Ltd. A5 Bldg, FuZhong Industrial Zone, Xia Shi Wei road, FuYong district, BaoAn, ShenZhen, China.	imported & distributed by medisana GmbH Carl-Schurz-Str. 2 41460 NEUSS GERMANY
--	--

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ShangHai International Holding Corp. GmbH (Europe)